


# WatchPAT™ ONE

## STEP BY STEP MANUAL

### WatchPAT™ ONE box contains:

1. Step by Step Manual
2. WatchPAT™ ONE device - with Finger probe (1) and Chest sensor (2)\*
3. AAA battery (3)

\*Chest sensor is not included in WatchPAT™ ONE E

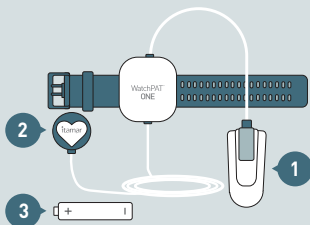

### Please read the following instructions before applying the WatchPAT ONE:

Download the WatchPAT ONE application to your mobile device [available on Apple App Store or Google Play Store].

**Note:** Make sure you have your four digits PIN to activate the device.

- Remove tight clothes, rings, watches, and other jewelry.
- Remove nail polish and artificial nails from the test finger and make sure the fingernail is cut short.
- The device should be worn on the non-dominant hand, and the finger sensor can be attached to any finger, except the thumb.

**Note:** Adult supervision may be required to apply the device.

### Operating the device:

#### Preparing to start the test:

- Please turn off all other transmitting devices in the room, such as smart watches, other smart phones and headphones, as it could interfere with the test.
- Open WatchPAT ONE application on your mobile device.
- Follow the instructions in the application that will guide you through the steps of operating the test.

**Note:** Your phone must have access to the internet while you are setting up the WatchPAT ONE device for the test.

#### At the end of the test:

1. Press the **END RECORDING** key in the WatchPAT ONE application and follow the instructions.
2. If the application is closed open it, make sure your phone is connected to the Internet and wait for the **"Thank You"** notification.

**Note:** Only when you see a **"Thank You"** message presented on the application, the test is completed and sent to your doctor.

## Technical Information Labeling



Caution: Federal Law Restricts this device to sale by or on the order of a licensed healthcare practitioner



Single use, do not re-use



Temperature limit 0°C - 40°C



Refer to instruction for use

1.5V DC

Battery operating voltage

IP22

Ingress protection



Type BF applied part



Federal communications commission identification number



Medical device manufacturer



Catalogue number



Serial number



Date of manufacture



Use-by date



The product is marked with the CE logo 2797 for BSI



Authorized representative in the European Community



According to the WEEE Directive 2012/19/EU, all waste electrical and electronic equipment (EEE) should be collected separately and not disposed of with regular household waste. Please dispose this product and all of its parts in a responsible and environmentally friendly way.



Fragile, Handle with care



Keep dry



### Important Notes:

- Storage and operation temperatures 0°C - 40°C
- Do not attempt to connect or disconnect any part of the unit.
- Do not try to introduce any foreign object into the unit.
- Do not, under any circumstances, attempt to fix a problem by yourself.



**Itamar Medical Ltd.**

9 Halamish Street, PO box 3579  
Caesarea Ind. Park, 3088900 Israel  
T: +972-4-6177000, F: +972-4-6275598



**Arazy Group GmbH**

The Squire 12, Am Flughafen  
60549 Frankfurt am Main  
Germany

**itamar**  
medical

[www.itamar-medical.com](http://www.itamar-medical.com)

REF: 0M2192390 - Ed.1