

THE POWER OF HEALING IN 3 RESORBABLE FORMS

Protect the Oral Wound Bed and Aid Healing
with ACE Resorbable Collagen Products.



resorbable collagen plug, foam & tape

- Provides an excellent matrix to assist in wound healing
- Beneficial in protecting the oral wound site
- Controls minor bleeding due to the intrinsic hemostatic properties of collagen fibers



RESORBABLE COLLAGEN PLUG, FOAM AND TAPE WOUND DRESSINGS

- Provides an excellent matrix to assist in wound healing
- Beneficial in protecting the oral wound site
- Controls minor bleeding due to the intrinsic hemostatic properties of collagen fibers
- Sterile

Features	Plug	Foam	Tape
Fully resorbable	✓	✓	✓
Ideal for applications in oral wounds	✓	✓	✓
Provides an excellent matrix to assist in wound healing	✓	✓	✓
Protects the oral wound bed	✓	✓	✓
Ideal for extraction sockets	✓		
Controls minor bleeding due to the intrinsic hemostatic properties of collagen fibers	✓	✓	✓



resorbable collagen plug



resorbable collagen foam



resorbable collagen tape

RC Plug	Regular	Member	Premier	Premier+
5094000 1cm x 2cm 10/box	\$129.49	\$129.49	\$110.07	\$110.07
RC Foam				
5092000 2cm x 4cm x 3mm 10/box	\$142.79	\$142.79	\$121.37	\$121.37
RC Tape				
5091000 2.5 x 7.5cm x 1mm 10/box	\$194.79	\$194.79	\$165.57	\$165.57

CREST PRESERVATION OF INTACT ALVEOLUS

Recommended by Dr. Aron Gonshor, PhD, DDS, FRCD(C), FAO



Atraumatic tooth extraction without flap reflection. Note: Maintain intact bony walls of the extraction socket and preserve gingival anatomy.




Alveolar socket is grafted to crest with xenograft bone mineral (NuOss®).



A resorbable collagen plug (RCP) is cut to size and compressed in place as a dressing.



Criss-cross and/or mattress sutures are placed.

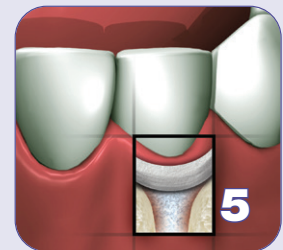

A removable or fixed prosthesis with an ovate pontic extending 2-3mm subgingivally is placed. This permits plug compression and tissue support.

Copyright © 2023 ACE SOUTHERN

RC Plug, Foam & Tape are manufactured for ACE Surgical

All prices are subject to change without notice. Prices shown are domestic prices in U.S. dollars; does not include shipping or any duties. For discount details visit acesouthern.com/ASAP. And for the most up-to-date pricing, please contact ACE SOUTHERN or visit our website.

The intended use of Resorbable Collagen Plug is for the management of oral wounds and oral sores. This document is intended exclusively for dental professionals in particular and expressly not for the information of laypersons. The information on the products and/or procedures contained in this document is of general nature and does not represent medical advice or recommendations. Since this information does not constitute any diagnostic or therapeutic statement with regard to any individual medical case, individual examination and advising of the respective patient are absolutely necessary and are not replaced by this document in whole or in part.